# Pic Health Station

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# EN App User Manual

## Index

1. Introduction	25
2. Download and install the Pic Health Station app	25
3. Register, Login and Logout	25
3.1 New user registration	25
3.2 Login and Logout	26
4. Menu and dashboard	27
4.1 Dashboard & timeline	27
4.2 Settings	28
4.2.1 My Profile	28
4.2.2 Devices	28
4.2.3 Measurement Types	32
4.2.4 Units	33
4.2.5 Temperature - Fever	33
4.2.6 Weight - My BMI	34
4.2.7 Pressure - Guidelines	34
4.2.8 Glycaemia - Guidelines	35
4.2.9 Video Tutorial	36
4.2.10 About	36
5. Blood Pressure Data Management	37
6. Weight Data Management	38
7. Temperature Data Management	40
8. Glycaemia Data Management	41
9. Cloud	
10. Multi-profile	44

## 1. INTRODUCTION

To measure, save, rank and share.  
It's easy with Pic!

The Pic Health Station app allows an easy and constant monitoring of your blood pressure values, your glycaemia\*, your weight and your child's fever. Easy to download and sync, Pic Health Station allows you to:

- Store your values automatically
- View your data at any time
- Perform an average of systolic and diastolic blood pressure values.
- Create a personal journal of your pressure, glycaemia\*, weight and temperature that is simple and up-to-date and share it with your trusted doctor.

## 2. DOWNLOAD AND INSTALL THE PIC HEALTH STATION APP.

You can download and install Pic Health Station from App Store for iOS and Google Play for Android.

## 3. REGISTER, LOGIN AND LOGOUT.

### 3.1 New user registration

Push the Register Button (Figure 1). Enter your email address and chose your password. Then push "Register" (Figure 2). An e-mail will be sent to your e-mail address by MedM since the Pic Health Station app is Powered by MedM.

You can also have access to your data through PC at the <https://health.medm.com> web-portal you can access your data from your PC as well.

In "My Profile" page you can enter your profile data: name, surname, date of birth and gender and height. The information about your height is important to correctly calculate the Body Mass Index in the weight module (Figure 3).

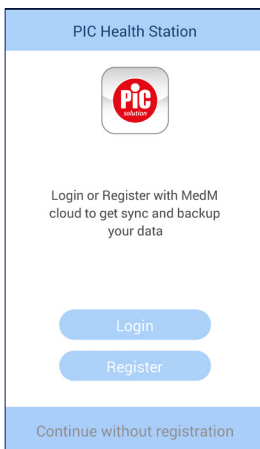
Figure 1

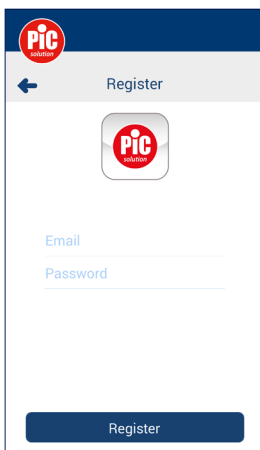
Figure 2

\*The glycaemia module is only visible if one has a Pic GlucoTest Diary, BeGlic or GoGlic blood glucose meter

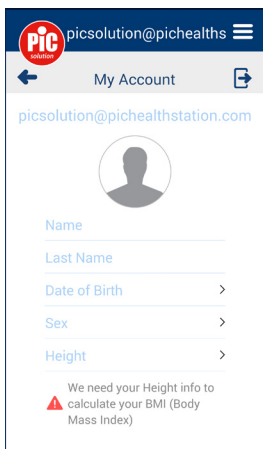
Figure 3

### 3.2 Login and Logout

Once you registered, you can enter your email address and password, then push “Login” (Figure 4) to start using the app.

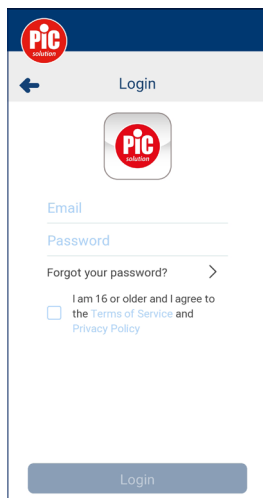
Figure 4

Note. If you forgot your password, enter your email address and push “Forgot your password?” to easily reset your password.

Note. It is not mandatory to register an account to use the Pic Health Station app. In case you don’t register the data will be stored only locally on your smartphone and you will not have the server back-up of your data.

To logout, push the menu icon , then choose “My Profile” and push the specific icon .

## 4. MENU AND DASHBOARD

### 4.1 Dashboard & timeline

In the  (Figure 5) tab you have an overview of the most recent measurement received from your devices. You may have one, such as blood pressure, or more than one, for example blood pressure, weight, temperature and glycaemia\*, depending on the app configuration. The list of measurement types will be automatically update when you synchronize a new device with the Pic Health Station app.

Figure 5

Select the type of measurement, weight for example to see previous measurements, the data trend pressing the  icon, share them pressing the  icon. Select a single testing to view more detailed information, to add notes or to delete the entry.

Figure 6

Select  (Figure 6) tab to check all the measurements sorted by time. You may click on each single measurement to get further details.

\*The glycaemia module is only visible if one has a Pic GlucoTest Diary, BeGlic or GoGlic blood glucose meter

## 4.2 Settings

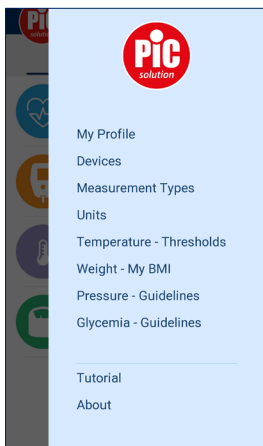
Figure 7

### 4.2.1 My Profile

Push the menu icon  Then choose "My Profile" to edit your profile info. Confirm with "V" or discharge with "X".

### 4.2.2 Devices

Push the menu icon  and choose 'Devices'.

### Add a new blood pressure monitor (mobileRAPID).

In order to synchronize a new blood pressure monitor mobileRAPID click on the "Add New" (Figures 9,10) button and hold the User2/Pairing button of the blood pressure monitor for 3 seconds, until you see two squares blinking up and down:  and . Tap on the device found and set your preferences (Figures 11, 12):

- Available for: if you choose "My Profile"

all data stored in mobileRAPID are sent to the logged profile in the Pic Health Station. "Any guest profile": only real time data acquisition mode. The app receives only the measurement you have just done and assigns it to the selected profile. No data already stored in mobileRAPID are sent to the Pic Health Station.

- User Memory: If you will be the only user of the device, select "all areas", otherwise select the user that you prefer (User 1  1 / User 2  2 ). Please note that if you chose the User area 1, you should start the measurement by pushing the BPM User 1 key, otherwise the blood pressure monitor and the app will not synchronize; if you chose the User area 2, you should start the measurement by pushing the BPM User 2 key, otherwise the blood pressure monitor and the app will not automatically synchronize.
- If you chose all areas you can use all BPM user keys to start the measurements and transfer them to the app.

Figure 8  
BPM User Keys

Confirm and complete the connection through the button "Add to my devices". If any error doesn't occur, you will see your mobileRAPID in your devices list. Otherwise try to repeat the procedure.

## Add a new blood pressure monitor (mobileRAPID WRIST).

- Turn on the smartphone/tablet Bluetooth® connection and Pic Health Station app.
- In the Pic Health Station menu, select “devices” and “add device”
- Press  **1** or  **2** on your MobileRAPID Wrist.
- The app will automatically detect your MobileRAPID Wrist. Follow the App instruction to complete the pairing.
- During the pairing, only the smartphone/tablet to be connected with the device shall run PIC APP. If you meet problem in pairing please check that no other smartphone in the family are running the APP, or reboot your smartphone.

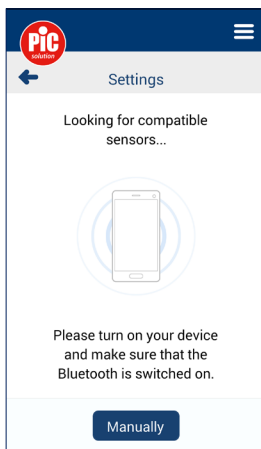
Figure 10

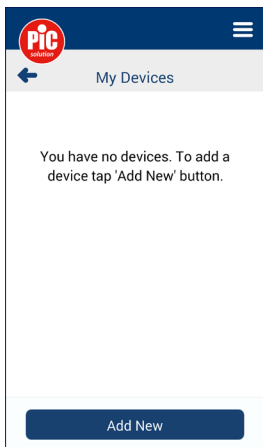
Figure 9

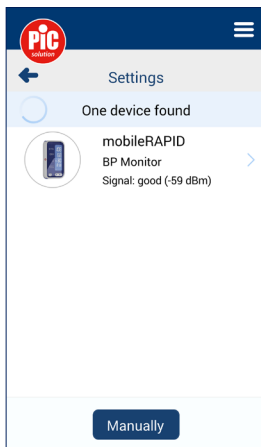
Figure 11

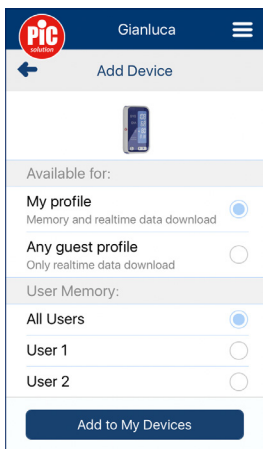
Figure 12

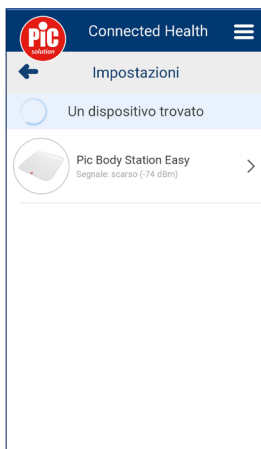
Figure 14.1

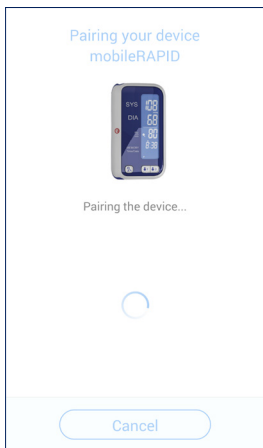
Figure 13

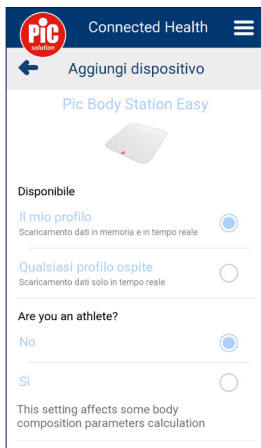
Figure 14.2

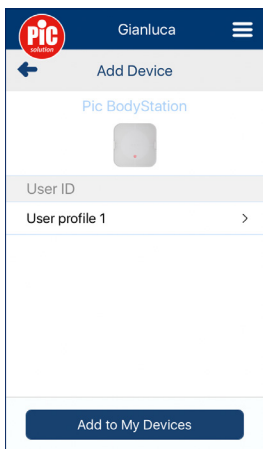
Figure 15

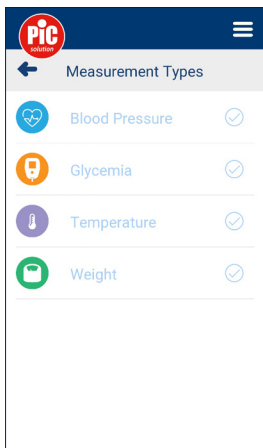
Figure 16

## Adding a new scale

### Add a new scale

In order to synchronize a new scale Pic BodyStation click on the “Add New” button and hold the button behind the scale (UNIT) button for 3 seconds, until you see two squares blinking up and down:  and . Tap on the device found and set your preferences.

- User Memory: Select your user memory from P1 to P8.

Confirm and complete the connection through the button “Add to my devices”. If any error doesn’t occur, you will see your Pic BodyStation in your devices list (Figure 15). Otherwise try to repeat the procedure.

### Add Pic BodyStation

To sync a new Pic BodyStation Easy scale, click the “Add new” button. Turn on the scale, pressing lightly on the surface, touch the device detected by the app (Figure 14.1) and set your preferences, specifying: 1) if you are the only scale user by selecting “My profile”, in this case the app will save all data stored in the scale’s memory and the last reading taken, or if the scale is used by multiple users by selecting “Any guest profile”, in this case the app will save only real time data; 2) if you are an athlete (Figure 14.2).

Confirm and complete the connection by pressing the “Add to my devices” button. If the procedure is successful, your Pic BodyStation Easy will appear in the device list. If errors occur, try to repeat the procedure.

### Add a new thermometer

To synchronize a new infrared thermometer ThermoDiary Head and/or ThermoDiary Ear click on the “Add New” button and turn on your ThermoDiary Head or ThermoDiary Ear (Figure 17). Tap on the device found. If any error doesn't occur, you will see your ThermoDiary Head or ThermoDiary Ear in your devices list. Otherwise repeat the procedure and/or reboot your smartphone.

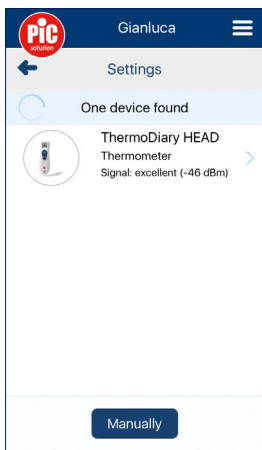
Figure 17

### Adding a new blood glucose meter

To synchronise a new blood glucose meter, click on the “Add new” button (Figure

9, 10). Follow the instructions below on the basis the meter being used.

### Pic Glucotest Diary

If you have a Pic Glucotest Diary blood glucose meter, turn on the meter pressing the ON/OFF button for 3 seconds until the “bt” icon appears on the screen and blinks, lower right hand corner. Select the device detected (Figure 18) and click on “Add to my devices”. If the meter is added correctly, the symbol “OK bt” will appear on the display of your Pic Glucotest Diary, and it will be visible in the list of devices. If any errors occur, turn off the meter and try repeating the procedure.

### BeGlic

To synchronise the BeGlic blood glucose meter you will need a BeGlic Bluetooth Transmitter. The BeGlic Bluetooth Transmitter may be requested by calling the consumer's toll-free number 800 900 080 for Italy. With the blood glucose meter off, insert the Bluetooth transmitter in the test strip port. Make sure that the Bluetooth indicator light is facing up before insertion. The text 'BLE' will appear on the display and the blue light on the transmitter will start to blink. The blue Bluetooth light will then remain on, indicating that the connection has been established. Select the device detected (Figure 18) and click on “Add to my devices”. At this point a message will appear on your smartphone/ tablet asking you to enter a PIN code. For the code, use the final six figures (excluding the last one) of the serial code, which is printed on the label located on the back of your BeGlic blood glucose meter. To complete the synchronisation, enter the PIN code as requested by the message that appeared

on the app. If the procedure is completed successfully, your BeGlic meter will appear in the list of devices. Otherwise, turn off the blood glucose meter and try repeating the procedure.

## GoGlic

To activate the Bluetooth function, turn on the GoGlic meter and press ▼ until the text 'BLE' appears on the display. The blood glucose meter's Bluetooth indicator, top left, will begin to blink and the display will remain blank, indicating that Bluetooth is active. Select the device detected (Figure 18) and click on "Add to my devices". A message will appear on your smartphone / tablet asking you to enter a PIN code. For the code, use the final six figures (excluding the last one) of the serial code, which is printed on the label located on the back of your GoGlic blood glucose meter. To complete the synchronisation, enter the PIN code as requested by the message that appeared on the app. If the procedure is completed successfully, your GoGlic meter will appear in the list of devices. If any errors occur, turn off the meter and try repeating the procedure.

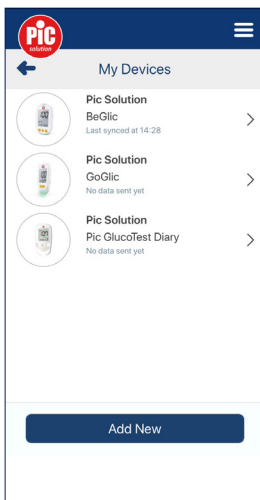
Figure 18

### Information from a device

Select a device from the list to see more detailed information. If the list is empty repeat the procedure used to add a new device.

### Delete a device

If you want to remove a device from your devices list, select it and push the "Delete" button . Then confirm to complete the operation.

In case of problems concerning settings, maintenance or use of the device, contact your local distributor.

### 4.2.3 Measurement Types

Push the menu icon  and choose "Measurement Types" to select the measurement types according to your device.

Choose “Blood Pressure” for the Blood Pressure Monitor, “Weight” for the Scale and “Temperature” for Thermometers. Deselect the measurement types unwanted (Figure 18).

#### 4.2.4 Units

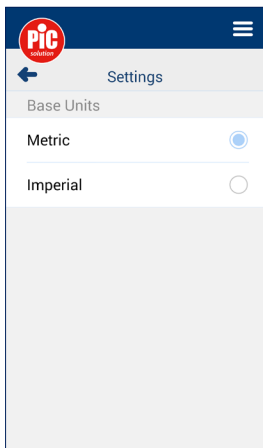
Figure 19

#### 4.2.5 Temperature - Fever

This section shows you current value for family fever alarm. Swipe right or left to set your preference (Figure 20).

Figure 20

#### 4.2.6 Weight - My BMI

This section shows your current weight and BMI classified accordingly the World Health Organization's table (Figure 21).

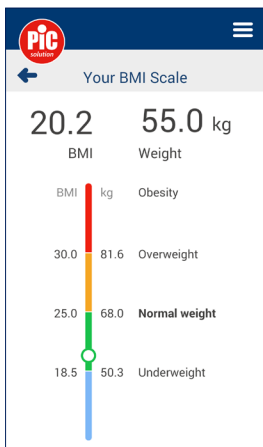
Figure 21

#### 4.2.7 Pressure – Guidelines

This section shows the 2013 ESH/ESH classification of blood pressure measured out of office (and this is the case of the self-measurement at home).

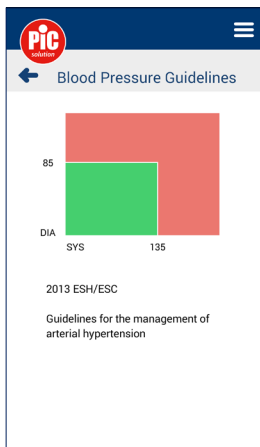
Figure 22

#### 4.2.8 Glycaemia - Guidelines

This section lists the main guidelines that specify what blood glucose levels should be achieved (Figure 23).

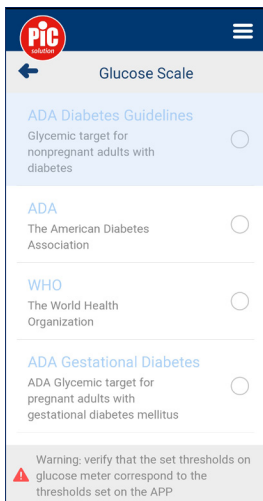
Figure 23

The guidelines used in your country can be selected by clicking next to the desired item. By selecting one of the guidelines you can see the preprandial and postprandial blood sugar targets. When viewing a given guideline, clicking on < or > will allow you to scroll through the various guidelines.

The glycaemia targets can be personalised manually from the "Custom" section. Press the  icon (Figure 24) to set the minimum and maximum limits Before a Meal and After a Meal (Figure 25). Input the desired values and press "Save" to apply the changes.


Figure 24


Figure 25

## 4.2.9 Apple Health

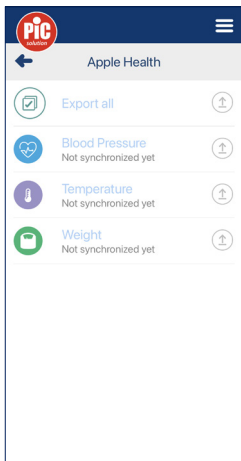Pic Health Station is compatible with Apple's Health application. If you have an iPhone, you can synchronize all your measurements stored in the App Pic Health Station, in the Apple Health App.


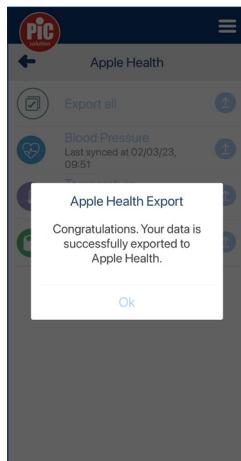
To migrate data, click on the menu icon and select Apple Health; once re-directed to the following screen, select the types of measurements you want to export or select Export All, if you want to migrate all the types of measurements displayed.

a screen will appear confirming the successful migration;

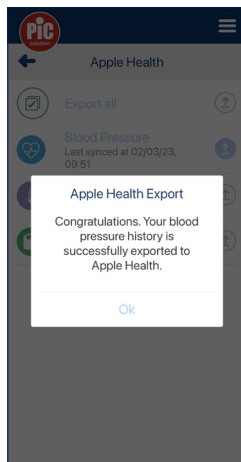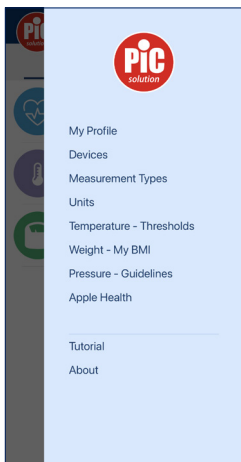in the event of an error, access the iPhone Settings, select the Health app, select "Data access and devices" and inside it enable the Pic Health Station parameters you want to export.

If you repeat the data migration following a new installation of the Pic Health Station App, you risk duplicating the data already exported.

Please note that the data transmitted to Salute is managed in compliance with the parameters guaranteed by the application itself and does not fall within the MedM treatment.

#### 4.2.10 Video Tutorial

In this section you can find some interesting video about how to use the blood pressure monitor and some video infographics for your health.

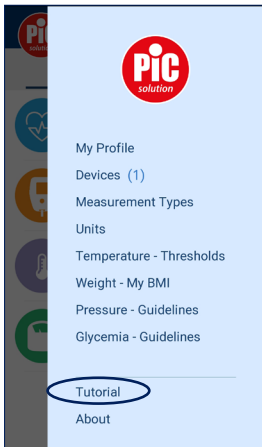
Figure 26

#### 4.2.11 About

Select "About" to check the app version and Privacy Policy.

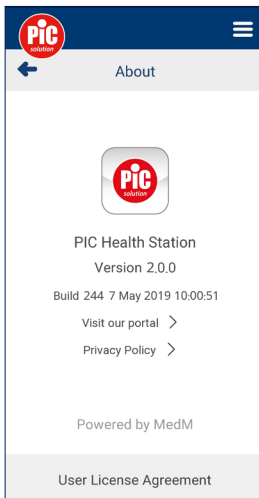
Figure 27

## 5. BLOOD PRESSURE DATA MANAGEMENT

Push on the “Blood Pressure” icon  to open blood pressure history. Note! If you activated only the blood pressure monitor module you will see directly the blood pressure Main page with the list of measurements (Figure 24).

To pair the blood pressure monitor and your smartphone, see the section “My devices”. After that you can measure your blood pressure on the Pic mobileRAPID blood pressure monitor and your data will be directly transferred to the app.

The reading appears in Pic Health Station screen and the trend graph is automatically updated. If your blood pressure monitor has historical readings stored, all of them will be uploaded as well.

You can check the blood pressure details by pressing the blood pressure data (Figure 24).

Push in the right upper corner of Blood Pressure page to check the chart of blood pressure in week and month.

The heart rate icon  allows you to see the heart rate values trend (Figure 25) and the blood pressure icon  allows you to come back to your pressure (systolic and diastolic) trend (Figure 26). You can check the weight trend by clicking on the trend icon .

Push the icon  to add a blood pressure data or symptoms, notes and drugs and the edit icon  to remark it manually. Only data added manually can be changed and remarked. Data uploaded from blood pressure monitor can only be remarked (Figure 27).

Figure 28

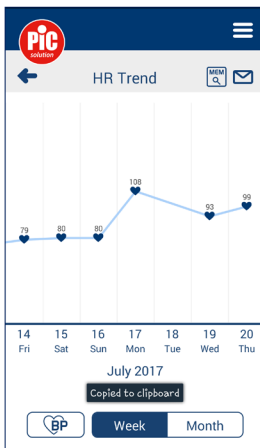
Figure 29

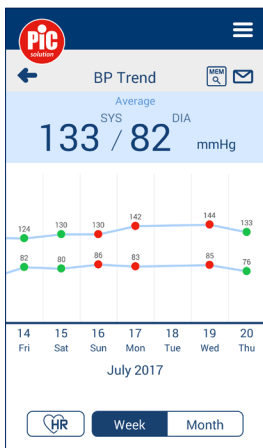
Figure 30

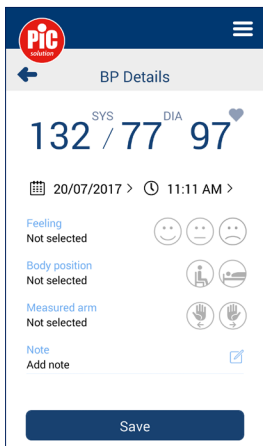
Figure 31

## 6. WEIGHT DATA MANAGEMENT

Press on the “Weight” icon  to open the weight section (Figure 28).

Measure your weight on the scale. Note! For body composition analysis you have to step on them without footwear to allow to calculate and transfer all the parameters.

Wait until the measure is done and the reading appears in Pic Health Station screen. In case of use of Pic BodyStation, if your weight scales have historical readings stored, all of them will also be uploaded. You can check the weight trend by clicking on the weight data .

(Figure 33) and History (Figure 32).

On weight detail page (Figure 34), you can edit a measurement or delete it by pushing  or . Note that only data added manually can be changed and remarked, instead data uploaded from the scale can only be remarked (Figure 35).

Click on the measurements list of Weight page to check the chart of weight in Week, Month and 3 Months.

Push the button  to add a weight data or symptoms, notes and drugs and remark manually.

Push “Add Goal”  on Weight page to set the weight goal (Figure 36). You can choose by which date you want to achieve your goal and therefore the difficulty level (Easy/Medium/Hard/Aggressive/Custom).

Click “Save” to record your goal settings.

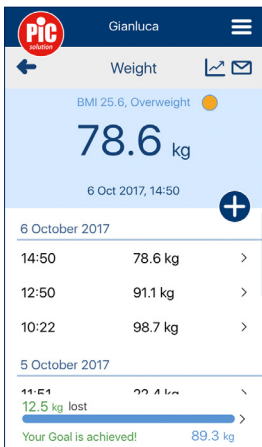
Figure 32

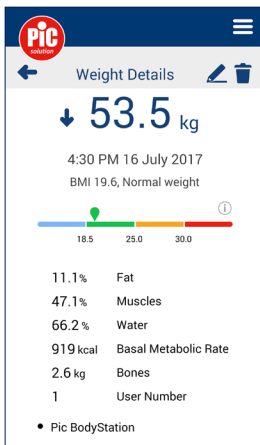
Figure 34

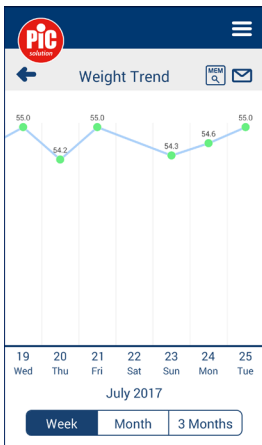
Figure 33

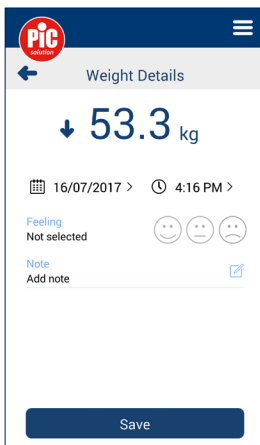
Figure 35

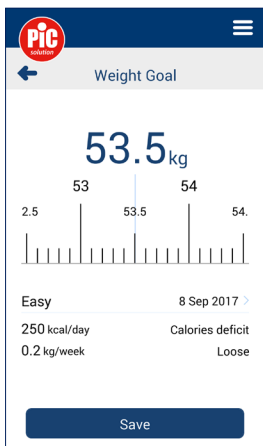
Figure 36

Figure 37

## 7. TEMPERATURE DATA MANAGEMENT

Press on the “Temperature” icon to open the temperature section (Figure 33). Measure your or your children’s temperature with the thermometer. Wait until the measure is done and the reading appears in Pic Health Station screen.

Note that the thermometers send data only when Pic Health Station is on on your smartphone.

Please take note that data stored in the memory of the thermometer are not sent to the app.

You can check the temperature trend by clicking on the trend icon .

On temperature detail page, you can edit a measurement or delete it by pushing .

Note that only data added manually can be changed and remarked, instead data uploaded from the scale can only be remarked.

By clicking “Add” icon you can add manual temperature, symptoms, drugs or notes and they will be displayed in temperature homepage. Please take note that all texts you will type in drug and symptoms pages will be saved and listed to be available for next insertions [for instance, all the time you will type in a new content in drug page, all drugs already used will be displayed and can be selected].

## 8. GLYCAEMIA DATA MANAGEMENT

Press the “Glycaemia” icon to open the glycaemia section.

Note: the glycaemia module is visible in these situations:

- if it has been activated in the “Measurement Types” section;
- after having synchronised the blood glucose meter;
- if the user has already saved glycaemia

data on the portal (accessible from Menu>About-Visit our portal).

In the event that the glycaemia module is not visible, remove the blood glucose meter from the "Devices" section and repeat the synchronisation.

Note: if you have only activated the glycaemia module in the "Measurement Types" section, the main glycaemia page will open directly with the list of measurements (Figure 38).

### Testing and transmitting the data

Measure your glycaemia with the blood glucose meter. The transmission of APP data will vary depending on the device used.

#### Pic Glucotest Diary

The Pic GlucoTest Diary blood glucose meter synchronises with the App:

- automatically when a test strip is removed from the meter after testing, if the APP is active
- manually by pressing the ON/OFF button on the meter for 3 seconds when the APP is active.

#### BeGlic

Once the BeGlic Bluetooth Transmitter is inserted in the test strip port of BeGlic blood glucose meter, the connection is made automatically, provided that the App is active.

#### GoGlic

The GoGlic blood glucose meter connects to the app in two different ways.

- at the end of testing mode, the Bluetooth indicator will show a fixed blue light and the display on the meter will remain blank, provided the app is active.
- with the blood glucose meter turned

off, by pressing ▼ while the app is active.

Note: once a connection has been established, the time and date on the GoGlic meter will automatically synchronise with the time and date on the smartphone/tablet.

For all Pic GlucoTest Diary, BeGlic and GoGlic devices, the reading appears on the Pic Health Station screen and the trend graph is automatically updated.

Note: if past testings have been stored on the blood glucose meter, these will be uploaded as well.

### Viewing and changing data.

You can see the glycaemia measurements by opening the History (Figure 38) of previous measurements.

Press the + icon and "Manual Glycaemia" to manually add a glycaemia measurement and "Symptoms," "Drug," "Note" to manually add a comment.

Press on a measurement to see the details (Figure 39): you may only change a glycaemia measurement if it was entered manually or delete it if it was either entered manually or transferred from the blood glucose meter (Figure 40). The details (date, time, feeling, meals and note) may always be changed manually.

Note: any text entered in the drugs and symptoms pages is saved and listed to allow for subsequent entries.

Press the ↗ icon to view the "Sequential Glycaemia" graph (Figure 41) for the given period - 1 day, 3 days, 1 week. To switch from "Sequential Glycaemia" view to "Overlapped Glycaemia" press the icon. You may choose to view 1 week, 15 days, 1 month (Figure 42).

## Measurement icons key

Every measurement may be associated with a moment of the day. The following options are available (Figure 43):

- “No details” (icon ●) if the testing was not paired with any marker (before a meal, after a meal, fasting, snacks, bedtime);
- “Before a meal” (whole apple icon):
  - 🍏 if the value is below target,
  - 🍏 if the value is on target,
  - 🍏 if the value is above target,
- “After a meal” (eaten apple icon):
  - 🍏 if the value is below target,
  - 🍏 if the value is on target,
  - 🍏 if the value is above target,
- “Fasting” (crossed out apple icon):
  - 🍏 if the value is below target,
  - 🍏 if the value is on target,
  - 🍏 if the value is above target,
- “Snacks” (yoghurt icon):
  - 🍏 if the value is below target,
  - 🍏 if the value is on target,
  - 🍏 if the value is above target.
- “Bedtime” (icon 🌙).

In addition, the following icons may appear:

⚠ when the testing is beyond the limits measurable by the meter (<10 mg/dL and >600 mg/dL).

CS when the glycaemia test refers to a test conducted using the control solution.

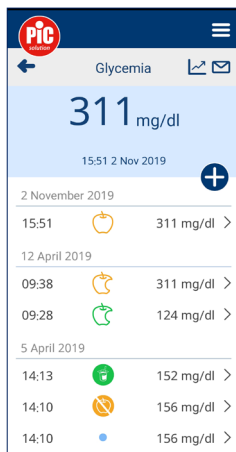
Figure 38

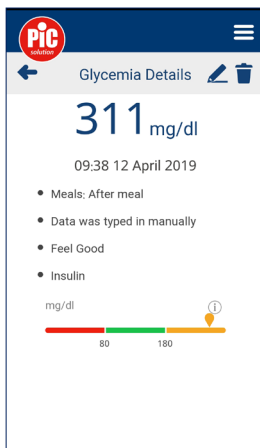
Figure 39

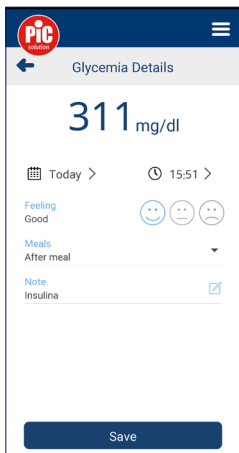
Figure 40

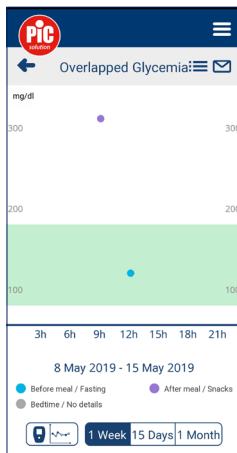
Figure 42

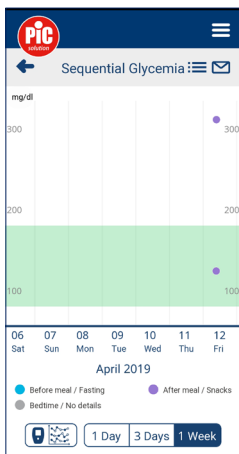
Figure 41

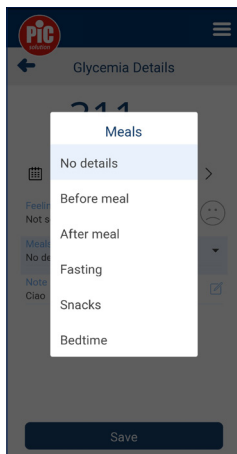
Figure 43

Figure 44

Figure 45

## 9. CLOUD

If you create an account with Pic Health Station, the data will be stored locally on your smartphone and on the cloud. You can access to your data from any smartphone/tablet where Pic Health Station is installed or from any PC from the website to <https://health.medm.com>.

## 10. MULTI-PROFILE

With Pic Health Station you can visualize or acquired data for more than one profile (from now ahead "guest profile") on your smartphone. For example, in this way you can check health conditions of all persons you care. To do it please follow these steps.

1. Each person who wants to share data with you must create his/her own profile, by registering himself/herself with Pic Health Station app [see paragraph 3].
2. Then he/she has to go to <https://health.medm.com> and sign in the website with the same e-mail and password used for Pic Health Station.
3. He/she must click on "Care Circles" and on the button "Invite, send invitation by e-mail" and follow the instructions. He/she has to write your e-mail address and confirm by sending the invitation.
4. You will receive an e-mail in order to accept or reject the invitation coming from the guest profile.

From now ahead in Pic Health Station you can switch among your profile and any guest profile activated (Figure 34, 35).

Warning:

1. The shown BMI index is the BMI index of the selected profile.
2. ThermoDiary: the thermometers al-

low data download to any profile.

3. mobileRAPID: the blood pressure monitor allow the data download as you set up during pairing process ["My profile" or "Any guest profile"].

Pic BodyStation: the scale allow the data download to the selected profile, not to "any guest profile", whose you can only visualize data without downloading them.

Pic GlucoTest Diary, BeGlic, GoGlic: the blood glucose meter only allows the data to be downloaded in the selected profile, not in "any guest profile". Data from a guest profile, if present and if you are authorised for access, may be viewed or submitted manually but downloading is not permitted. Data may be uploaded by the guest using his/her own devices.

- The Bluetooth® trademark and logos are registered trademarks of SIG. Inc. and any use thereof by PIKDARE S.p.A. is under license.
- Google Play and the Google Play logo are Google Inc. trademarks.
- Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and in other countries. App Store is a service trademark of Apple Inc. registered in the United States and in other countries.
- The app layout may change depending on its release. Phone not included.
- The blood pressure, temperature and glycaemia modules of the Pic Health Station APP are Class I medical devices compliant with Regulation (EU) 2017/745, and, as such, are NOT devices meant for direct diagnosis.
- We wish to emphasize that the App

and services are only to be used to as a personal health diary to keep history and build awareness but they cannot replace consultation with and diagnosis by a healthcare professional. Under no circumstances does the App issues medical therapy recommendations or medical advice of any kind. Always seek the advice of your physician or other qualified healthcare professionals with any questions you may have regarding a medical condition, diet, fitness or wellness program. No content – no matter whether provided by the App itself, our cooperation partners, or users – can be understood as supplementing or replacing information from a healthcare professional or pharmacist.

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- A Google Play és a Google Play logó a Google Inc. védjegyei.