Pic Health Station

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-  Denis Khitrov (ecrep@medm.com)
Leopoldstr 136, Munich, 80804, DE

Index

1. Introduction	20
2. Download and install the Pic Health Station app	20
3. Register, Login and Logout	20
3.1 New user registration	20
3.2 Login and Logout	21
4. Menu and dashboard	22
4.1 Dashboard & timeline	22
4.2 Settings	23
4.2.1 My Account	23
4.2.2 My devices	23
4.2.3 Measurement Types	26
4.2.4 Units	27
4.2.5 Weight - My BMI	27
4.2.6 Pressure - Guidelines	28
4.2.7 Temperature - Thresholds	28
4.2.8 Video tutorial	29
4.2.9 About	29
5. Blood Pressure Data Management	30
6. Weight Data Management	31
7. Temperature data management	33
8. Cloud	34
9. Multi-profile	34

1. INTRODUCTION

To measure, save, rank and share.
It's easy with Pic!

The Pic Health Station app allows an easy and constant monitoring of your pressure values values, of your weight and of your child's fever.

Easy to download and sync, Pic Health Station allows you to:

- Store your values automatically
- View your data at any time
- Perform an average of systolic and diastolic blood pressure values.
- Create your personal journal of your pressure, weight and temperature, simple and up-to-date and share it with your trusted doctor.

2. DOWNLOAD AND INSTALL THE PIC HEALTH STATION APP.

You can download and install Pic Health Station from App Store for iOS and Google Play for Android.

3. REGISTER, LOGIN AND LOGOUT.

3.1 New user registration

Push the Register Button (Figure 1).

Enter your email address and chose your password. Then push "Register" (Figure 2). An e-mail will be sent to your e-mail address by MedM since the Pic Health Station app is Powered by MedM.

You can also have access to your data through PC at the <https://health.medm.com> web-portal you can access your data from your PC as well.

In "My Profile" page you can enter your profile data: name, surname, date of birth and gender and height. The information about your height is important to correctly calculate the Body Mass Index in the weight module (Figure 3).

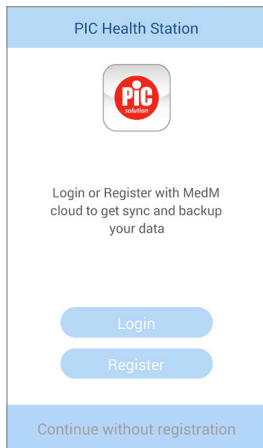
Figure 1

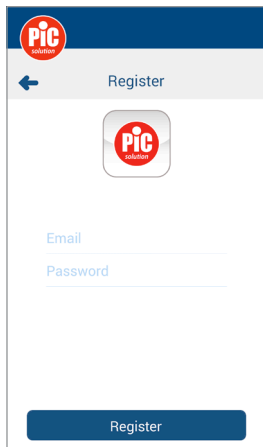
Figure 2

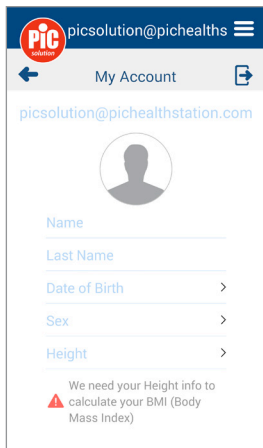
Figure 3

3.2 Login and Logout

Once you registered, you can enter your email address and password, then push "Login" (Figure 4) to start using the app.

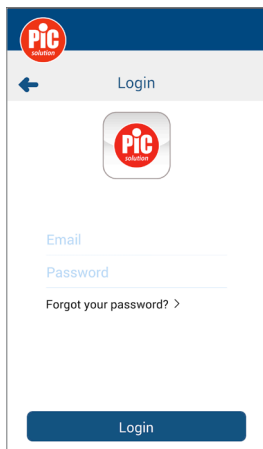
Figure 4

Note. If you forgot your password, enter your email address and push "Forgot your password?" to easily reset your password.

Note. It is not mandatory to register an account to use the Pic Health Station app. In case you don't register the data will be stored only locally on your smartphone and you will not have the server back-up of your data.

To logout, push the menu icon  then choose "My Profile" and push the specific icon .

4. MENU AND DASHBOARD

4.1 Dashboard & timeline

In the  (Figure 5) tab you have an overview of the most recent measurement received from your devices. You may have one, such as blood pressure, or more than one, for example blood pressure and weight, depending on the app configuration. The list of measurement type will be automatically update when you synchronize a new device with the Pic Health Station app.

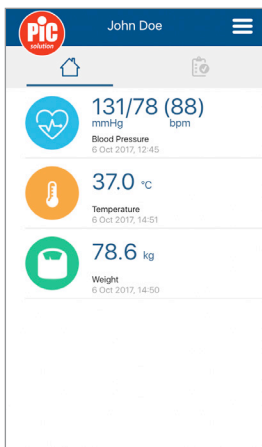
Figure 5

You may click the single measurement to enter in the section where you can see the data trends, share by email , have access to details or add notes to your measurements.

Figure 6

Select  (Figure 6) tab to check all the measurements sorted by time. You may click on each single measurement to get further details.

4.2 Settings

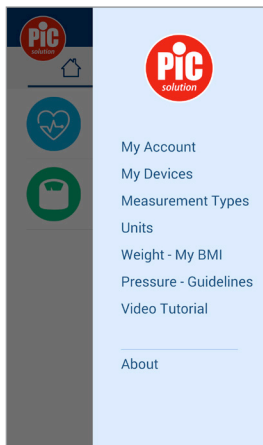
Figure 7

4.2.1 My Profile

Push the menu icon . Then choose "My Profile" to edit your profile info. Confirm with "V" or discharge with "X".

4.2.2 My devices

Push the menu icon  and choose 'Devices'.

Add a new blood pressure monitor.

In order to synchronize a new blood pressure monitor mobileRAPID click on the "Add New" (Figures 9,10) button and hold the User2/Pairing button of the blood pressure monitor for 3 seconds, until you see two squares blinking up and down:  and . Tap on the device found and set your preferences (Figures 11, 12):

- Available for: if you choose "My Profile" all data stored in MobileRapid are sent

to the logged profile in the Pic Health Station. "Any guest profile": only real time data acquisition mode. The app receives only the measurement you have just done and assigns it to the selected profile. No data already stored in MobileRapid are sent to the Pic Health Station.

- User Memory: If you will be the only user of the device, select "all areas", otherwise select the user that you prefer (User 1  1 / User 2  2). Please note that if you chose the User area 1, you should start the measurement by pushing the BPM User 1 key, otherwise the blood pressure monitor and the app will not synchronize; if you chose the User area 2, you should start the measurement by pushing the BPM User 2 key, otherwise the blood pressure monitor and the app will not automatically synchronize (figure 13).
- If you chose all areas you can use all BPM user keys to start the measurements and transfer them to the app.

Figure 8
BPM User Keys

Confirm and complete the connection through the button "Add to my devices". If any error doesn't occur, you will see your mobileRAPID in your devices list. Otherwise try to repeat the procedure.

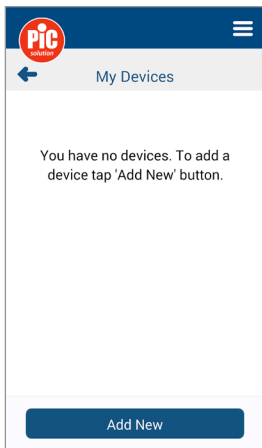
Figure 9

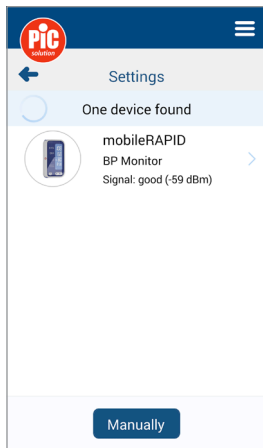
Figure 11

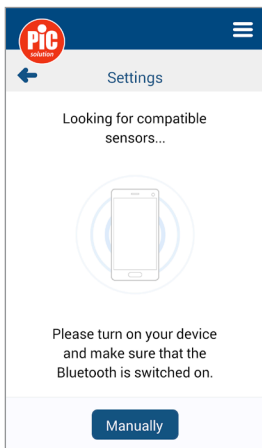
Figure 10

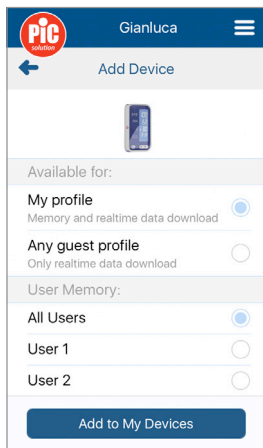
Figure 12

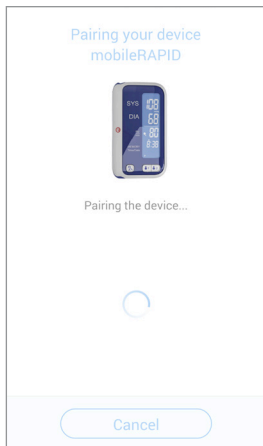
Figure 14

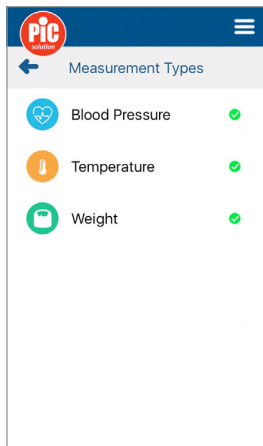
Figure 16

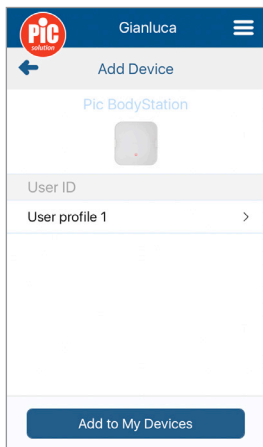
Figure 15

Add a new scale

In order to synchronize a new scale Pic BodyStation click on the "Add New" button and hold the button behind the scale (UNIT) button for 3 seconds, until you see two squares blinking up and down:  and . Tap on the device found and set your preferences (Figure 14).

- User Memory: Select your user memory from P1 to P8.

Confirm and complete the connection through the button "Add to my devices". If any error doesn't occur, you will see your Pic BodyStation in your devices list (Figure 15). Otherwise try to repeat the procedure.

Add a new thermometer

To synchronize a new infrared thermometer ThermoDiary Head and/or ThermoDiary Ear click on the "Add New" button and turn on your ThermoDiary Head or ThermoDiary Ear (Figure 17). Tap on the device found.

If any error doesn't occur, you will see your ThermoDiary Head or ThermoDiary Ear in your devices list. Otherwise repeat the procedure and/or reboot your smartphone.

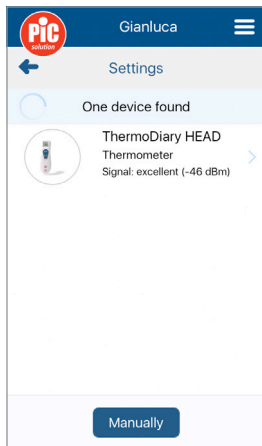
Figure 17

Delete a device

If you want to remove a device from your devices list, select it and push the "Delete" button .

Then confirm to complete the operation.

In case of problems concerning settings, maintenance or use of the device, contact your local distributor.

4.2.3 Measurement Types

Push the menu icon  and choose "Measurement Types" to select the measurement types according to your device.

Choose "Blood Pressure" for the Blood Pressure Monitor, "Weight" for the Scale and "Temperature" for Thermometers. Deselect the measurement types unwanted (Figure 18).

4.2.4 Units

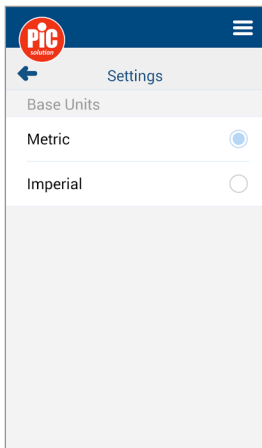
Figure 18

4.2.5 Weight – My BMI

This section shows your current weight and BMI classified accordingly the World Health Organization's table (Figure 19).

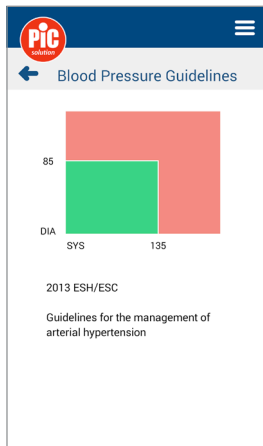
Figure 19

4.2.6 Pressure – Guidelines

This section shows the 2013 ESH/ESH classification of blood pressure measured out of office (and this is the case of the self-measurement at home).

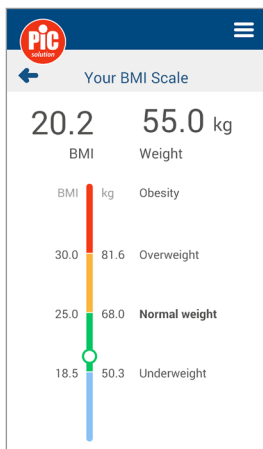
Figure 20

4.2.7 Temperature - Thresholds

This section shows your current value for family fever alarm. Swipe right or left to set your preference (Figure 21).

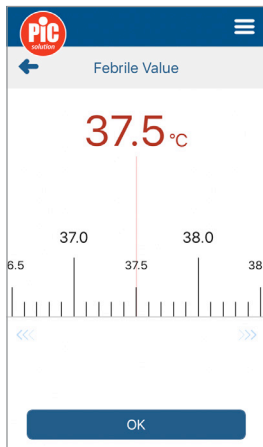
Figure 21

4.2.8 Video tutorial

In this section you can find some interesting video about how to use the blood pressure monitor and some video infographics for your health.

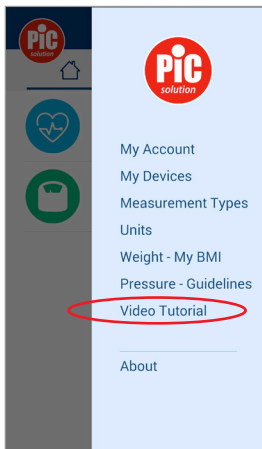
Figure 22

4.2.9 About

Select "About" to check the app version and Privacy Policy.

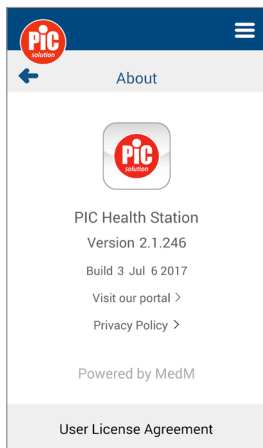
Figure 23

5. BLOOD PRESSURE DATA MANAGEMENT

Push on the “Blood Pressure” icon  to open blood pressure history. Note! If you activated only the blood pressure monitor module you will see directly the blood pressure Main page with the list of measurements (Figure 24).

To pair the blood pressure monitor and your smartphone, see the section “My devices”. After that you can measure your blood pressure on the Pic mobileRAPID blood pressure monitor and your data will be directly transferred to the app.

The reading appears in Pic Health Station screen and the trend graph is automatically updated. If your blood pressure monitor has historical readings stored, all of them will be uploaded as well.

You can check the blood pressure details by pressing the blood pressure data (Figure 24).

Push in the right upper corner of Blood Pressure page to check the chart of blood pressure in week and month.

The heart rate icon  allows you to see the heart rate values trend (Figure 25) and the blood pressure icon  allows you to come back to your pressure (systolic and diastolic) trend (Figure 26). You can check the weight trend by clicking on the trend icon .

Push the icon  to add a blood pressure data or symptoms, notes and drugs and the edit icon  to remark it manually. Only data added manually can be changed and remarked. Data uploaded from blood pressure monitor can only be remarked (Figure 27).

Figure 24

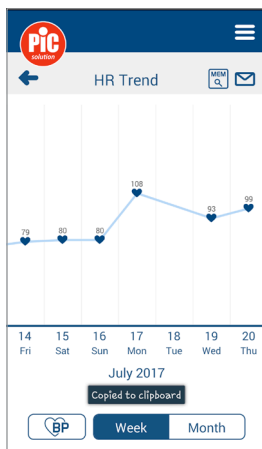
Figure 25

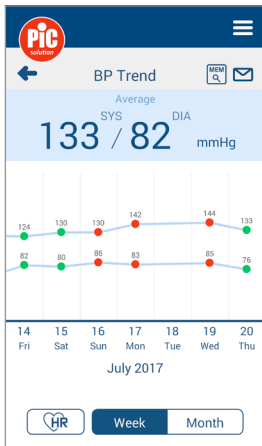
Figure 26

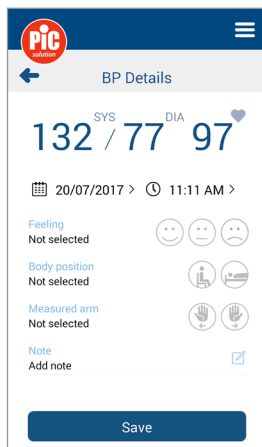
Figure 27

6. WEIGHT DATA MANAGEMENT

Press on the “Weight” icon  to open the weight section (Figure 28).

Measure your weight on the scale. Note! For body composition analysis you have to step on them without footwear to allow to calculate and transfer all the parameters.

Wait until the measure is done and the reading appears in Pic Health Station screen. If your weight scales have historical readings stored, all of them will also be uploaded. You can check the weight trend by clicking on the weight data .

(Figure 29) and History (Figure 28).

On weight detail page (Figure 30), you can edit a measurement or delete it by pushing  or . Note that only data added manually can be changed and remarked, instead data uploaded from the scale can only be remarked (Figure 31).

Click on the measurements list of Weight page to check the chart of weight in Week, Month and 3 Months.

Push the button  to add a weight data or symptoms, notes and drugs and remark manually.

Push “Add Goal”  on Weight page to set the weight goal (Figure 32). You can choose by which date you want to achieve your goal and therefore the difficulty level (Easy/Medium/Hard/Aggressive/Custom).

Click “Save” to record your goal settings.

Figure 28

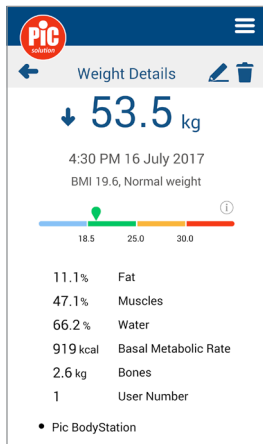
Figure 30

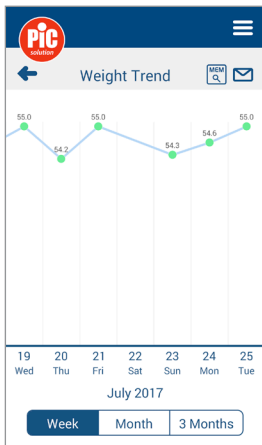
Figure 29

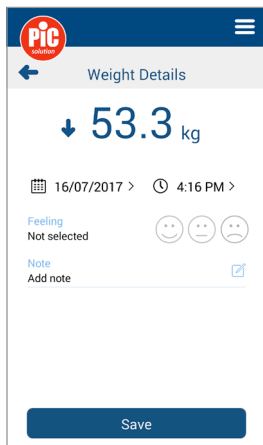
Figure 31

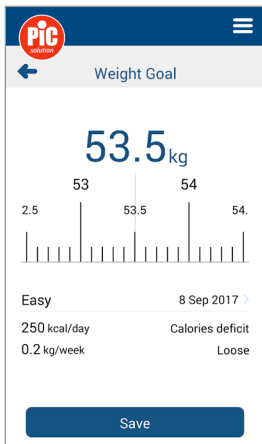
Figure 32

Figure 33

7. TEMPERATURE DATA MANAGEMENT

Press on the "Temperature" icon to open the temperature section (Figure 33). Measure your or your children's temperature with the thermometer. Wait until the measure is done and the reading appears in Pic Health Station screen.

Note that the thermometers send data only when Pic Health Station is on on your smartphone.

Please take note that data stored in the memory of the thermometer are not sent to the app.

You can check the temperature trend by clicking on the trend icon .

On temperature detail page, you can edit a measurement or delete it by pushing or .

Note that only data added manually can be changed and remarked, instead data uploaded from the scale can only be remarked.

By clicking "Add" icon you can add manual temperature, symptoms, drugs or notes and they will be displayed in temperature homepage. Please take note that all texts you will type in drug and symptoms pages will be saved and listed to be available for next insertions [for instance, all the time you will type in a new content in drug page, all drugs already used will be displayed and can be selected].

Figure 34

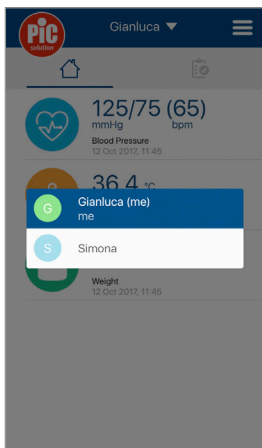
Figure 35

8. CLOUD

If you create an account with Pic Health Station, the data will be stored locally on your smartphone and on the cloud. You can access to your data from any smartphone/tablet where Pic Health Station is installed or from any PC from the website to <https://health.medm.com>.

9. MULTI-PROFILE

With Pic Health Station you can visualize or acquired data for more than one profile (from now ahead “guest profile”) on your smartphone. For example, in this way you can check health conditions of all persons you care. To do it please follow these steps.

1. Each person who wants to share data with you must create his/her own profile, by registering himself/herself with Pic Health Station app [see paragraph 3].
2. Then he/she has to go to <https://health.medm.com> and sign in the website with the same e-mail and password used for Pic Health Station.
3. He/she must click on “Care Circles” and on the botton “Invite, send invitation by e-mail” and follow the instructions. He/she has to write your e-mail address and confirm by sending the invitation.
4. You will receive an e-mail in order to accept or reject the invitation coming from the guest profile.

From now ahead in Pic Health Station you can switch among your profile and any guest profile activated (Figure 34, 35).

Warning:

1. The shown BMI index is the BMI index of the selected profile.
2. ThermoDiary: the thermometers al-

low data download to any profile.

3. MobileRapid: the blood pressure monitor allow the data download as you set up during pairing process ["My profile" or "Any guest profile"].

Pic BodyStation: the scale allow the data download to the selected profile, not to "any guest profile", whose you can only visualize data without downloading them.

- The Bluetooth® trademark and logos are registered trademarks of SIG. Inc. and any use thereof by PIKDARE S.r.l. is under license.
- Google Play and the Google Play logo are Google Inc. trademarks.
- Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and in other countries. App Store is a service trademark of Apple Inc. registered in the United States and in other countries.
- The app layout may change depending on its release. Phone not included.
- The Pic HealthStation app blood pressure and temperature modules are class I CE medical device not to be used for direct diagnosis.
-  The Pic HealthStation app blood pressure module is compliant to the European Medical Device directive 93/42/EEC, amended by the 2007/47/CE.
- Manufacturer:

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- Google Play e o logotipo Google Play são marcas registadas, propriedade da Google Inc.
- Distribuidor: PIKDARE S.r.l., Via Saldarini Catelli 10, 22070 - Casnate con Bernate (CO) - Itália